---

**DENTSPLY**  
DETREY

**AH Plus™**

**Root Canal Sealing Material**

|                                  |           |
|----------------------------------|-----------|
| <b>Directions for Use</b>        | <b>2</b>  |
| <b>Gebrauchsinformation</b>      | <b>8</b>  |
| <b>Mode d'emploi</b>             | <b>14</b> |
| <b>Gebruiksaanwijzing</b>        | <b>20</b> |
| <b>Istruzioni per l'uso</b>      | <b>26</b> |
| <b>Instrucciones para el uso</b> | <b>32</b> |
| <b>Instruções de utilização</b>  | <b>38</b> |
| <b>Bruksanvisning</b>            | <b>44</b> |
| <b>Brugsanvisning</b>            | <b>50</b> |
| <b>Instrukcja użytkowania</b>    | <b>56</b> |
| <b>Návod k použití</b>           | <b>62</b> |
| <b>Használati utasítá</b>        | <b>68</b> |

**DENTSPLY**  
DETREY

AH Plus™

**Root Canal Sealing Material**

AH Plus™ is a two-component paste:paste root canal sealer based on epoxy-amine resin chemistry. This easy-to-mix sealer adapts closely to the walls of the prepared root canal and provides minimal shrinkage upon setting as well as outstanding long-term dimensional stability and sealing properties.

AH Plus complies with the requirements of ISO 6876:1986 (E) for dental root canal filling materials.

## **COMPOSITION**

### **AH Plus Paste A**

Epoxy resins  
Calcium tungstate  
Zirconium oxide  
Silica  
Iron oxide pigments

### **AH Plus Paste B**

Amines  
Calcium tungstate  
Zirconium oxide  
Silica  
Silicone oil

## **INDICATION**

Permanent obturation of root canals of teeth of the secondary dentition in combination with root canal points.

## **CONTRAINDICATION**

Hypersensitivity against epoxy resins, amines or other components of the root canal filling material.

## **WARNINGS**

AH Plus contains epoxy resins or amines which may cause sensitisation in susceptible persons.

## **PRECAUTIONS**

Do not use AH Plus in persons allergic to epoxy resins or amines.

Avoid contact of single pastes or unset mixed paste with skin or oral mucosa. After incidental contact, wash and rinse with plenty of water.

Wear protective gloves and glasses.

## **INTERACTIONS WITH OTHER DENTAL MATERIALS**

None known.

## **ADVERSE REACTIONS**

With sealers containing epoxy resins, the following adverse reactions were reported:

- Reversible acute inflammation of the oral mucosa after contact with the unset paste.
- In individual cases, local and systemic allergic reactions have been reported.

## **STEP BY STEP INSTRUCTIONS**

### **Preparation of the Root Canal**

Prior to application of AH Plus, prepare, clean and dry the root canals to be filled using state-of-the-art endodontic techniques.

## Dosage and Mixing

Mix equal volume units (1:1) of Paste A and Paste B on a glass slab or a mixing pad using a metal spatula<sup>1</sup>. Mix to a homogeneous consistency.

**Note:** Under certain storage conditions, AH Plus Paste B may slightly separate. It has been shown that this process does not negatively affect the performance of the mixed product.

**Caution:** Do not exchange caps of tubes. The coloured cap belongs to the coloured tube, the white cap belongs to the white tube.

## Application

As a standard technique, AH Plus is used in combina-

tion with guttapercha, silver or titanium root canal points.

Apply the mixed AH Plus onto the tip of a Lentulo spiral. Avoid the formation of air bubbles in the material and overfilling of the canal by advancing the Lentulo spiral slowly to the apex running at very low speed. Withdraw Lentulo very slowly still running at low speed. Alternatively, AH Plus may be applied by using reamers. Choose a reamer of the size of the last instrument used during apical root canal preparation. Place AH Plus by a pumping action of the reamer with simultaneous rotary movement in a counterclockwise direction.

Dip the disinfected and dry master point into AH Plus and with pumping motions slowly push it into the canal.

---

<sup>1</sup> Mixing ratio by weight is 1 g Paste A to 1.18 g Paste B.

Additional guttapercha points and lateral or vertical condensation methods can be applied. In this case, apply only a light coating of AH Plus.

### **Removal of Root Canal Filling**

If AH Plus is used in combination with guttapercha points, the root canal fillings can be removed using conventional techniques for the removal of guttapercha.

### **Working Time**

The working time is at minimum 4 hours at 23 °C<sup>2</sup>.

### **Setting Time**

The setting time is at minimum 8 hours at 37 °C<sup>2</sup>.

---

<sup>2</sup> measured according to ISO 6876:1986 (E).

### **CLEANING OF INSTRUMENTS**

Spatulas, mixing slabs and instruments should be cleaned immediately after use with alcohol or acetone.

### **STORAGE**

Store at room temperature.

Keep resin tubes tightly closed.

### **BATCH NUMBER AND EXPIRY DATE**

The batch number should be quoted in all correspondence which requires identification of the product.

Do not use after expiry date.

Patent No.: US 217998; EP 0673637

If you have any questions, please contact:

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© DENTSPLY DeTrey 2002-03-01
**DENTSPLY DeTrey GmbH**