---

**DENTSPLY**  
DETREY

## AutoMatrix®

**Retainerless Matrix System**

**Halterloses Matrizensystem**

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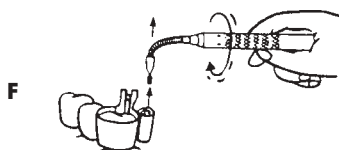
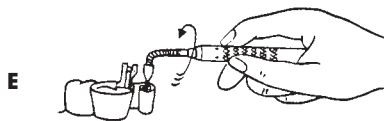
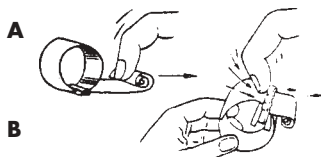
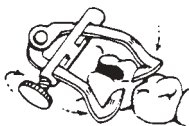
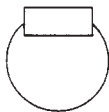
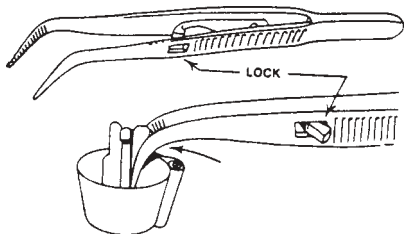
## Retainerless Matrix System

### Introduction

There are four types of **AUTOMATRIX** bands as explained in the table below. All sizes fit teeth of any circumference. Only the height and gauge of the bands are different.

Type	Medium/ thin	Narrow/ regular	Medium/ regular	Wide/ regular
Height (mm) Gauge (mm)	6.35 0.038	4.76 0.05	6.35 0.05	7.94 0.05
Clinical uses	bicuspid/ molars	short bicuspid and molars	bicuspid/ molars	long molars

### Enlarging matrix circumference

To properly fit tooth, matrix should be slightly larger than occlusal circumference.

1. Unroll coil slightly with fingernail ([fig. A](#)).
  2. Using fingernail, slide the AUTO-LOCK against the coil ([fig. B](#)).
- If necessary, repeat steps 1 and 2 above to further enlarge matrix.

### Positioning matrix

Position matrix with the AUTO-LOCK at the center of buccal or lingual or the remaining section of standing tooth ([fig. C](#)).

### Tightening the matrix

(using AUTOMATE III tightening device)

1. Until tightened, stabilize matrix to **gingival** with finger.  
Insert tip into coil (**DO NOT USE FORCE**) ([fig. D](#)).  
NOTE: AUTOMATE III has sonic torque control, thereby requiring only minimal rotational force.
2. Rotate knurled handle **CLOCKWISE**. After 2 or 3 **sonic** clicks, **STOP!**  
The band is subgingivally tightened ([fig. E](#)).  
NOTE: The AUTO-LOCK keeps matrix locked tightly even during the following manipulations.

### Removing the tightening device

Rotate AUTOMATE III device continuously **counterclockwise** (backwards) and **simultaneously and continuously** withdraw AUTOMATE III device from coil ([fig. F](#)).

### Placement of restorative

Place wedges as desired (buccal and/or lingual) ([fig. G](#)).

Burnish contact points. Condense amalgam and do rough carving.

### Removing the matrix

CAUTION: During any of the following matrix removal procedures a segment ([fig. H](#)) of the AUTO-LOCK LOOP (retention clip) may separate from the matrix band and fall into the mouth. To prevent this segment from being swallowed or aspirated, remove it immediately from the mouth.

To open and remove matrix, use shielded SNIPPERS and clip any part of projecting end of AUTO-LOCK LOOP ([fig. I](#)).

Keep SNIPPERS closed during withdrawal from mouth, then drop cut pieces on tray.

Insert a probe through LOCK-RELEASE HOLE ([fig. J](#)) located near the coil.

Gently pull LOCK-RELEASE HOLE occlusally until the matrix slips open ([fig. K](#)).

To avoid amalgam failures, slide matrix buccally or lingually, while removing matrix occlusally.

If segment of AUTO-LOCK LOOP has separated from band, remove it from mouth ([fig. L](#)).

### Sterilizing/sanitizing

The stainless steel **AUTOMATRIX** bands and the AUTOMATE III tightening device may be autoclaved or sterilized (dry heat sterilization 350 °F / 295 °C). The AUTOMATE III device should not be cold sanitized. The shielded Snippers should only be cold sanitized.

### Helpful Hints

- To seat matrix in tight interproximals – use an Elliot Separator on posteriors and gain temporarily required space ([fig. M](#)).
- To clear debris and facilitate seating – use waxed floss, or a soft wedge between tight interproximals or pass a Jelenko Separating Disk through ([fig. N](#)).
- For hard to reach molars – use locking College Pliers to grip matrix and facilitate seating ([fig. O](#)).

NOTE: The winding tip of the AUTOMATE III tightening device will readily release from the coil only if it is simultaneously and continuously wound backwards during withdrawal from the tightened coil.

If you have any questions, please contact:

**Manufacturer:**  
**DENTSPLY DeTrey GmbH**  
**De-Trey-Str. 1**  
**78467 Konstanz**  
**GERMANY**  
**Phone +49 (0) 75 31 5 83-0**

**Distributor:**  
**DENTSPLY Limited**  
**Hamm Moor Lane**  
**Addlestone, Weybridge**  
**Surrey KT15 2SE**  
**Phone (0 19 32) 85 34 22**

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© DENTSPLY DeTrey 1998-08