Degulux® soft-start

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Contents	Page
A. Packing list	8
B. Technical data	8
C. Health hazards	9
D. Safety precautions	9
E. Description of the product	10
F. Installation	10
G. Operation	10
H. Servicing and cleaning	11

A. Packing list

- 1 **Degulux® soft-start**
- 1 Light guide
- 1 Antiglare device
- 1 Operating instructions / Guarantee

B. Technical data

Power supply :	Refer to the rating plate for the voltage
Frequency:	50 Hz / 60 Hz
Power input:	110 W
Degulux Halogen lamp:	12 V / 75 W
Safety fuse for 110–120 V power supply: for 220–240 V power supply:	4 A Type T 2 A Type T
Classification:	Protection class II Protective system IP21 Application part - Type B
Mode of operation:	Short-time operation
Wavelength of the emitted light	400–500 nm
Mean light intensity at 400–500 nm:	Approx. 600–800 mW / cm ²
Dimensions:	
Base unit (H/W/D):	120 / 150 / 200 mm
Handpiece, incl. light guide:	170 / 55 / 240 mm
Weight	
Base unit	Approx. 2 kg
Handpiece:	Approx. 250 g
Light curing time:	soft-start (40 s), 20 s, 40 s or continuous operation
Operating conditions	+10°C – +35°C, Must be operated in dry areas only
Storage conditions	-15°C – +50°C, 5%–80% relative humidity, non-condensing
Beeps every 10 seconds	

Accessories:

Standard light guide, diameter 8 mm, length 90 mm (Order-No. 59221000)

Degulux® soft-start Halogen lamp 12 V / 75 W (Order-No. 59221050)

C. Health hazards - Must be read!!!

The **Degulux® soft-start** illuminates with extrem intensity. **Never aim the light directly at unprotected soft tissue (mucous membranes or gingiva)**, as this may cause injury or irritation.

Do not aim the light at eyes. Limit the action of the light to the area being treated.

During prolonged light curing procedures, light reflected from the tooth surface may injure eyes. Use the **antiglare device (eye protecting filter, order No.: 59221100)** supplied with the unit or **suitable safety glasses**, which shield effectively against blue spectrum light. **Sunglasses are not suitable!**

The **Degulux® soft-start** must not be used on patients, or by operators, with a heart pacemaker who have been advised to be cautious when using small electric appliances.

Anyone with a history of **retinopathy should consult their eye specialist** before operating this unit. Use the **Degulux® soft-start** extremely carefully and comply with all the necessary safety precautions (including wearing suitable, light filtering safety glasses).

Anyone who has had a **cataract operation** may be especially sensitive to light and should be advised against undergoing treatment with a **Degulux® soft-start** unless adequate safety precautions are taken such as wearing safety glasses which filter out blue/violet and ultra-violet light.

Do not use the **Degulux® soft-start** on patients with a history of photobiological reactions, (including patients with **Urticaria solaris** or **erythropoietic protoporphyrie**) or those currently undergoing treatment with photosensitizing **pharmaceuticals (including 8-methoxypsoralens or dimethyl chlorous tetracycline)**.

D. Safety precautions

Before installing this unit, read the operating instructions carefully. Always adhere strictly to these operating instructions.

We assume no liability whatsoever for damage caused due to this unit having been used for procedures other than those described in these operating instructions.

Before connecting this unit, check that the voltage range on the rating plate of the **Degulux® soft-start** coincides with the mains voltage where it is to be connected.

Do not open this unit. There are dangerously high voltages inside the base unit. Only authorized technicians are allowed to repair this unit.

Safeguard this unit against moisture to prevent short-circuiting and damage to it.

Only operate the **Degulux® soft-start** with original spare parts and accessories. This is the only way of attaining the performance quoted and ensuring safe operation.

This unit must not be used in an atmosphere which may lead to an explosion.

For safety reasons, the lamp may only be switched on with the cone fitted.

When replacing defective components as described in these operating instructions, only fit original DENTSPLY components. We assume no liability whatsoever for damage due to the use of other manufacturer's components.

To avoid electric shocks, do not insert objects into the unit - except for proper replacement of components as described in these instructions.

The unit must be switched off and unplugged before replacing the lamp. The lamp may be hot. Do not touch the lamp if it is hot.

If the unit is moved from a cold area to a warm area, condensation may create dangerous conditions. Therefore, the unit should not be operated until it has attained room temperature.

Install the unit where the air can circulate freely. The ventilation slots in the handpiece must not be covered.

If, for any reason whatsoever, it is assumed that the safety of this unit is impaired, cease operating it and mark it to prevent other people operating it inadvertently. This unit may be unsafe to use if, for example, it does not function as described or it has been damaged visibly.

Keep solvents, inflammable liquids and intense sources of heat away from this unit as they damage its plastic housing.

When cleaning this unit, ensure cleaning agents do not get into the unit, as this may cause electrical short-circuit or dangerous malfunction.

E. Description of the product

The **Degulux® soft-start** light curing unit is a source of high intensity, cold, visible blue light.

The 20 sec., 40 sec. and continuous light curing settings are for polymerizing dental materials in the patient's mouth which cure at a wavelength of 400 - 500 nm (e.g. **Definite®**).


The **Degulux® soft-start** also features a soft start light curing mode which cures at low intensity followed by an increase in the intensity.

The soft start mode reduces the stress in the material and is particularly indicated for **Definite®**.

The special 75 Watt Halogen lamp guarantees high lighting power throughout the entire service life of the Halogen lamp. An optical filter eliminates the ultraviolet and infrared radiation.

F. Installation

The **Degulux® soft-start** is a cabinet top unit.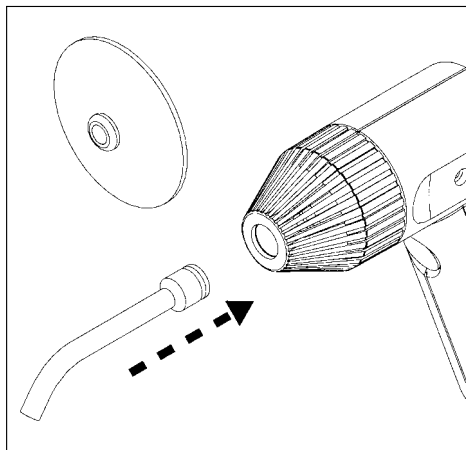

Before installing the unit, insert the light guide into the handpiece. The light guide should be inserted into the conical end of the handpiece and pressed in until it locks into place.

When not in use, the handpiece should be placed on the base unit.


Before connecting the unit, check that the voltage range quoted on the Degulux soft-start rating plate coincides with the mains voltage where it is to be connected.

Connect the power supply.

Switch the **Degulux® soft-start** on with the flip switch on the left of the base unit - it is then ready for use. The green lamp indicates that the unit is ready for use and shows the curing mode selected.

G. Operation

The light curing time is set by pressing the preset time button [soft start (40 secs.), 20 secs., 40 secs., continuous operation].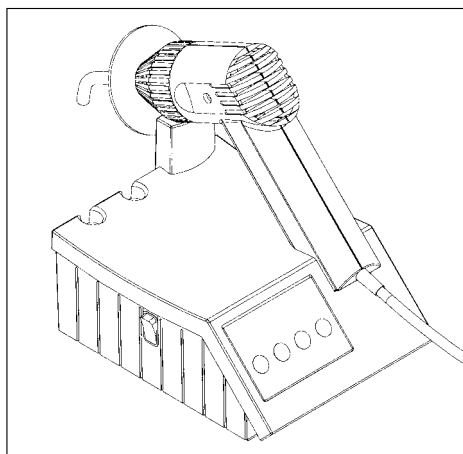

Press the black button on the handpiece to switch on the Degulux® soft-start. A beeper can be heard every 10 seconds while the unit is in operation.

If the soft-start mode has been selected, the unit commences polymerization at reduced lighting power, increases it slowly to maximum and switches the lamp off after 40 seconds.

If the time has been preset (20 secs., 40 secs.) the lamp switches off automatically once the selected time has elapsed.

Light curing can be terminated at any time by simply pressing the button on the handpiece again.

Depending on how long the unit was switched on, the cooling fan continues running after the lamp has been switched off. Do not switch the unit off at the power switch or unplug it while the fan is still running.

1. Polymerisation

Remove the handpiece from the holder and position the light guide close to the filling.

Switch the light on by pressing the button on the handpiece briefly. Avoid contacting unpolymerised filling material as it would polymerise on the light guide and impede light transmission.

Also avoid contacting cured filling material or bonding agent as a thin, reactive smear layer remains on the surface of cured filling material. (This layer ensures that the material bonds to composite or bonding agent perfectly).

The curing time is provided by the composite/bonding agent manufacturers' instructions.

Before operating the **Degulux® soft-start**, a photometer should be used regularly to check for adequate lighting power as, for example, cured filling material may adhere to the light guide (refer to section H "Maintenance").

2. Preventing overheating

Once the fan in the handpiece has cooled it properly, the fan switches off automatically.

If the Halogen lamp is on continuously for longer than 3 minutes or switched on intermittently with only brief intervals in between, the overheat cut-out may be activated. All 4 LEDs on the control panel of the unit flash to indicate that it is overheated.

After cooling for approximately 2 - 3 minutes, the **Degulux® soft-start** is ready for use again. Do not switch the unit off or unplug it while it is cooling.

H. Servicing and cleaning

1. Servicing

Check the light guide, housing components and supply lines regularly to ensure that they are intact. Contaminated or damaged light guides may impede the performance of the light considerably.

Repairs may only be carried out by authorized service technicians.

2. Cleaning

The **Degulux® soft-start** can be cleaned with a soft cloth moistened with an alcohol-based disinfectant. Do not use acetone or other strong solvents!

3. Disinfection/Sterilizing the light guide

Before use, the glass sheathed light guide must always be sterilized in steam at a temperature of up to 125°C.


The standard hygiene procedures for dental practices must be heeded.

4. Replacing the Halogen lamp


Only use an original **Degulux® soft-start** Halogen lamp as this is the only lamp which attains the performance quoted and polymerizes perfectly.

For safety reasons, the lamp must only be switched on with the cone fitted.

Switch the unit off, unplug it and allow the Halogen lamp to cool.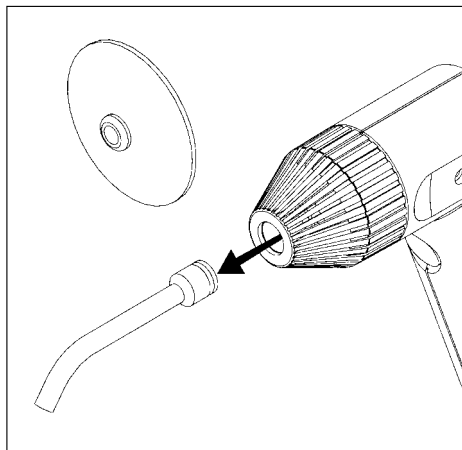


Remove the light guide

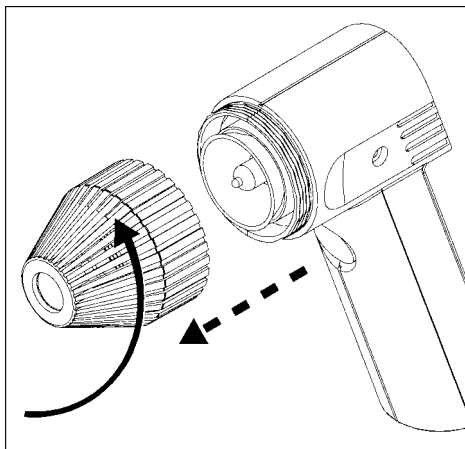Unscrew the cone from the handpiece anti-clockwise.

Grasp the **edge** of the Halogen lamp reflector and pull it out of the holder.

DO NOT TWIST IT !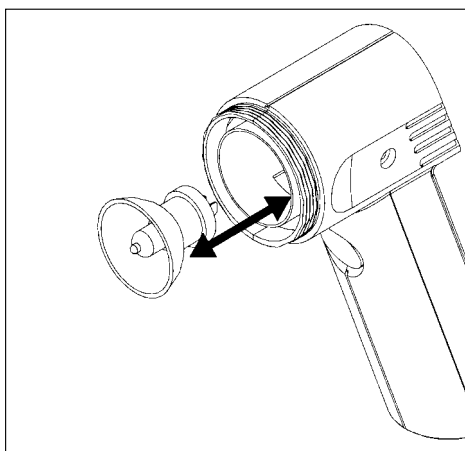

Grasp the new Halogen lamp on the edge of its reflector and press it into the holder to the stop.

DO NOT TOUCH THE INNER SURFACE !

The inner surface of the reflector and the bulb must be free of contamination and fingerprints before the unit is switched on. Contaminated areas can be cleaned with alcohol.


Wind the cone on clockwise.

Insert the light guide until it clicks into place.

5. Replacing the fuse

Unplug the unit from the mains.

The fuse is on the bottom of the unit.

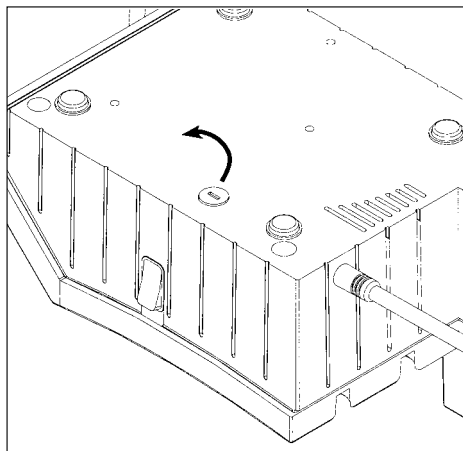Turn the fuse holder anti-clockwise with a screwdriver until the holder pops out.

Insert a new fuse into the fuse holder, press the holder into the unit and turn it clockwise with a screwdriver until it locks in .

Guarantee terms:

Every unit is tested thoroughly before being shipped.

This unit is guaranteed under the following terms:

The guarantee period begins on the date of delivery and expires after 12 months.

We will rectify free of charge defects due to faulty materials or craftsmanship which occur during the guarantee period. Additional claims outside of the minimum legal requirements for guarantees are ruled out.

Carriage charges, customs and import duties incurred in the customer's own country, must be met by the customer unless the law demands otherwise.

The guarantee is considered null and void if repairs or other procedures are carried out by anyone not authorized by us to do so. Claiming on a guarantee does not prolong the guarantee period.

Guarantee claims can only be accepted if accompanied by a **stamped** guarantee certificate **and** invoice or delivery note.

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DENTSPLY DeTrey GmbH
DENTSPLY Limited
DENTSPLY DeTrey GmbH, Konstanz