home Illuminé™

Tooth Whitening System

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Laboratory

Instructions for Tray Fabrication


Patient: _____

Dentist: _____

Date: _____



Whitening Technique:

- Illuminé home
- Illuminé office or Illuminé office & home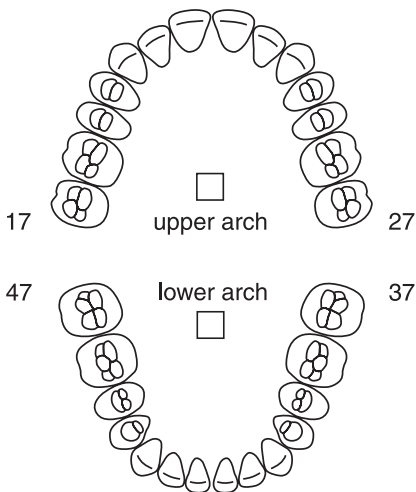

Teeth and arch to be treated are marked by a cross ~~X~~

Instructions for Tray Fabrication

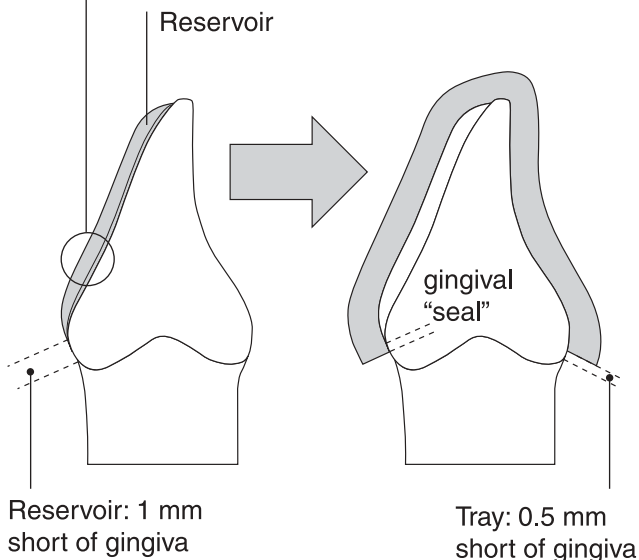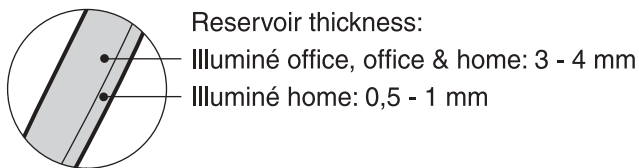Note: Gel reservoir is created only for teeth to be treated

Instructions for Tray Fabrication _____	2
Anleitung zur Herstellung der Tiefziehschiene _____	4
Instructions pour la fabrication du porte-empreinte _____	6
Richtlijnen voor de vervaardiging van de bleekbeugel ____	8
Istruzioni per la realizzazione della mascherina _____	10
Instrucciones para la fabricación de la cubeta _____	12
Instruções para a fabricação de moldeiras _____	14
Anvisning för framställning av gelskena _____	16
Instruktion for skinne fremstilling _____	18
Инструкции по изготовлению каппы _____	20
Instrukcja użytkowania dla laboratorium protetycznego __	22
Pokyny pro výrobu nosiče _____	24

English

Deutsch

Français

Nederlands

Italiano

Español

Português

Svenska

Dansk

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Polski

Česky

home Illuminé™

Tooth Whitening System

A well fitting tray with a proper gingival seal and margins just away from the gingiva is essential to reduce irritation of the gingiva during the whitening procedure to a minimum.

STEP-BY-STEP INSTRUCTIONS FOR MANUFACTURE OF THE WHITENING TRAY

Generate Stone

Pour the disinfected, rinsed impression with dental stone.

Trim the stone cast so that the base is parallel to the occlusal plane of the posterior teeth and that the base extends 2 - 4 mm past the gingival border.

Tray Fabrication

For tray fabrication, please see illustration on the last page of this booklet. Here, you will also find the dentist's instructions which teeth are to be treated.

Creating the Reservoirs

1. Once the stone model has dried to an effective hardness apply an acrylic spacer (e. g. Triad¹ Gel) to the teeth on the model, thus creating reservoirs in the gel tray.
2. Apply the spacer only to teeth of aesthetic value.
3. Stay approximately 1 mm from the gingival margin when applying the spacer.
4. Do not cover any interproximal areas.
5. Do not apply spacer to occlusal surfaces or incisal edges.
6. Apply a spacer to the facial surfaces of the teeth to be whitened. For treatment with Illuminé™ home tooth whitening gels alone, the thickness of the spacer is approx. 1/2 to 1 mm.
7. Wipe the cured spacer surfaces with a gauze dampened with alcohol to remove the slightly tacky surface that remains on the cured acrylic spacer.

Forming the Tray

1. Use a medium thick heat/vacuum sheet for tray fabrication.
2. Place the vacuum sheet (pebble finish towards the stone model) on a heat/vacuum tray-forming machine.
3. Soften the vacuum sheet until it sags 2 to 5 cm or as directed by the Manufacturer.
4. Engage the vacuum, and slowly lower the heated gel tray onto the stone model to avoid generating wrinkles and folds.
5. Allow the sheet ample time under vacuum for good retention and definition.
6. Allow the sheet to cool prior to trimming.

Trimming the Tray

1. Remove the gel tray from the stone model.
2. Using scissors to trim the tray approximately 2 - 3 mm beyond the teeth.

¹ DENTSPLY DeTrey, Dreieich (D).

3. Trim and refine the remaining portion with smaller precision scissors. (Scallop around the interdental areas.)
4. Trim the tray approximately 1/4 - 1/2 mm short of the gingival margin.
5. Place the trimmed tray back onto the stone model to verify extensions.
6. Finish margins with rotary instruments or slightly flame polish tray edges.
7. Wash and disinfect the whitening tray. Dry thoroughly.
8. Place in storage case or other suitable container and deliver to dentist.

If you have any questions, please contact:

Manufacturer:

DENTSPLY DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
GERMANY
Phone +49 (0) 75 31 5 83-0

Distributor:

DENTSPLY Limited
Hamm Moor Lane
Addlestone, Surrey
Weybridge KT15 2SE
Phone (0 19 32) 85 34 22

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© DENTSPLY DeTrey 2001-02-14