Directions for:

Zircate® Prophy Paste

Cleaning and Polishing Material

INDICATIONS

Zircate Prophy Paste, is indicated for use to effectively clean and pol-ish tooth structure with minimal abrasion.

Caution: U.S. Federal law restricts this device to sale by or on the order of a dentist.

COMPOSITION

Zircate Prophy Paste is composed of zirconium silicate in selected and controlled particle sizes in an aqueous spearmint flavored base.

CONTRAINDICATIONS

Zircate Prophy Paste is contraindicated in patients with a known aller-gy to any of the components.

WARNINGS

Zircate Prophy Paste may cause eye irritation upon contact.If contact with eyes occur, immediately flush eyes with copious quantities of water for 15 minutes. Seek medical attention if irritation or discomfort persists.

PRECAUTIONS

- All tooth surfaces should be thoroughly scaled to remove calculus.
- Necessary examination and radiographs should be taken prior to prophylaxis with Zircate Prophy Paste.
- During use, wear protective glasses and gloves.

STEPBY STEPINSTRUCTIONSFOR USE

- Dispense Zircate Prophy Paste into dappen dish or other suitable container. A saliva ejector may be helpful during the actual prophy laxis.

- Fill rubber prophylaxis cup with paste. Before starting handpiece, carry paste to field of operation and deposit a small amount on each tooth surface. It is necessary that fresh paste be provided for each surface in order to obtain maximum cleaning effectiveness.

- Run rubber cup at SLOWSPPEED over each tooth surface for approximately 10 seconds. Make sure that the edge of the cup goes below the free margin of the gingival into the gingival sulcus. Continue procedure until all exposed surfaces of the teeth have been thoroughly cleaned and polished.

- A rubber cup or soft bristle brush may be used on occlusal surfaces.

TECHNIQUE HINT

Use unwaxed dental floss to draw excess Zircate Prophy Paste through interproximal areas.

STORAGE

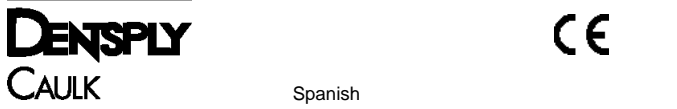
Replace cap on tube when not in use. Do not store paste above 25°C (77°F). Keep out of direct sunlight.

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All Rights Reserved
©1998 DENTSPLY International Inc.All rights reserved.
DENTSPLY International Inc.
DENTSPLY DeTrey GmbH, De-Trey Strasse 1, D-78467 Konstanz